
# Liver Assist Perfusion Set

**en:** Instructions for Use  
**de:** Bedienungsanleitung  
**fr:** Mode d'emploi  
**es:** Instrucciones de uso  
**it:** Istruzioni per l'uso  
**br:** Instruções de uso

**nl:** Gebruiksaanwijzing  
**pt:** Instruções de Utilização  
**dk:** Brugsanvisning  
**se:** Bruksanvisning  
**no:** Bruksanvisning

**it:** Naudojimo instrukcija  
**pl:** Instrukcja użycia  
**cz:** Návod k použití  
**ro:** Instrucțiuni de utilizare  
**tr:** Kullanım Talimatları

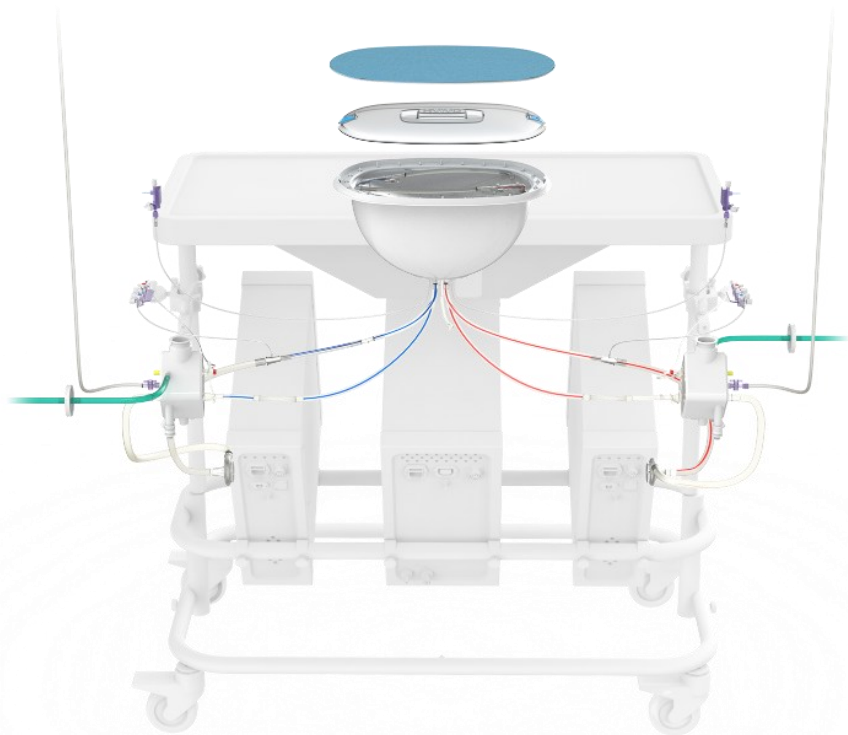# Content

<b>en:</b>	<b>Instructions for Use</b>	<b>2</b>
<b>de:</b>	<b>Bedienungsanleitung</b>	<b>3</b>
<b>fr:</b>	<b>Mode d'emploi</b>	<b>4</b>
<b>es:</b>	<b>Instrucciones de uso</b>	<b>5</b>
<b>it:</b>	<b>Istruzioni per l'uso</b>	<b>6</b>
<b>br:</b>	<b>Instruções de uso</b>	<b>7</b>
<b>nl:</b>	<b>Gebruiksaanwijzing</b>	<b>8</b>
<b>pt:</b>	<b>Instruções de Utilização</b>	<b>9</b>
<b>dk:</b>	<b>Brugsanvisning</b>	<b>10</b>
<b>se:</b>	<b>Bruksanvisning</b>	<b>11</b>
<b>no:</b>	<b>Bruksanvisning</b>	<b>12</b>
<b>lt:</b>	<b>Naudojimo instrukcija</b>	<b>13</b>
<b>pl:</b>	<b>Instrukcja użycia</b>	<b>14</b>
<b>cz:</b>	<b>Návod k použití</b>	<b>15</b>
<b>ro:</b>	<b>Instrucţiuni de utilizare</b>	<b>16</b>
<b>tr:</b>	<b>Kullanım Talimatları</b>	<b>17</b>

## en: Instruction for Use

### Liver Assist Perfusion Set

For oxygenated hypothermic and normothermic perfusion of donor livers

#### PRODUCT DESCRIPTION

##### Intended purpose

The Liver Assist Perfusion Set is intended to be used for ex-vivo hypothermic and normothermic oxygenated machine perfusion to preserve and evaluate donor livers prior to transplantation.

##### Patient Target Groups

The Liver Assist is (indirectly) intended for patients in need of a liver transplant.

##### Indications

The Liver Assist is indicated for use with deceased donor livers

##### Contraindications

There are no known contra-indications.

##### Intended Users

The Liver Assist is intended to be used in a clinical environment and operated by trained and licensed healthcare professionals familiar with medical practices required for organ perfusion. Users shall successfully complete a training course provided by XVIVO.

##### Clinical Benefit

The Liver Assist is intended to improve clinical outcomes for transplant recipients, enable otherwise marginal organs to be transplanted safely, thereby increasing the number of livers available for transplantation, and extend how long the liver can be preserved to allow more flexibility in the timing of the transplant operation.

##### Application Time

The Liver Assist is intended for hypothermic perfusion up to 24 hours and normothermic perfusion up to 6 hours.

##### Combination with Other Equipment

The Liver Assist Perfusion Set is only to be used in combination with the Liver Assist device (REF 11.101 & 11.01.101).

##### Sterile State

The Liver Assist Perfusion Set is delivered sterile (EO) and has a non-pyrogenic fluid path.

##### Information for Safety

Carefully read this user instruction, and the Instructions for Use for the Liver Assist before using the Liver Assist Perfusion Set.

##### Warnings and Precautions

- The product is sterile in the unopened and undamaged packaging. Carefully inspect the product for breaches of the integrity of the packaging. Do not use if the packaging or Liver Assist Perfusion Set is damaged, or even if there is the slightest doubt regarding the sterility of the product.
- Check the expiry date on the Liver Assist Perfusion Set. Do not use if the date has expired.
- The Liver Assist Perfusion Set is for single use only and may not be re-used or re-sterilized.
- XVIVO is not responsible for any damage caused operating the device outside the intended use.
- Modifications, especially post-treatments of the Liver Assist Perfusion Set is not allowed; they would affect safety and lead to exclusion of liability.
- Safe use of the Liver Assist can only be guaranteed when the operator is a skilled and trained professional and has successfully followed a Liver Assist training course.
- Any serious incident that has occurred in relation to Liver Assist should be reported to XVIVO and the competent authority of the Member State in which the user and/or patient is established.
- Do not tap the oxygenator and pump head with hard tools during priming procedure.
- Please contact XVIVO directly with any complaint at: qa.xn1@xvivogroup.com

##### Storage, Transport and Disposal

Do not expose the device to direct sunlight or strong artificial light

- Temperature: 10 - 30°C
- Humidity: 5 - 85 %RH non-condensing
- Atmospheric pressure: 50 kPa to 106 kPa

Shelf life: 3 years

Dispose the used Liver Assist Perfusion Set according to the valid regulations in your country.

### Contents of the Liver Assist Perfusion Set

1x Liver Assist Perfusion Set circuit	1x Y-connector
2x Filling line	1x Aortic cannula 24 Fr
2x Sample line	1x Portal cannula 24 Fr
1x Inner lid	1x Bile line
1x Second lid	1x Stepped male Luer lock connector

### Instruction - How to set up and operate

Read the IFU of the Liver Assist for complete use and guidelines for performance of a perfusion procedure.

Minimum operating volume: 2000 ml

Maximum operating volume: 4000 ml

### Serious Incidents

Note that any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the member state in which the user and/or patient is established.

### Symbols

	Consult the Instructions for Use
	Do not re-use
	Date of manufacture
	Use by date
	Batch code
	Catalogue number
	Sterilized using ethylene oxide
	Do not re-sterilize
	Manufacturer
	Temperature limit
	Humidity limit
	Do not use if the package is damaged
	Medical Device
	Single sterile barrier system
	Double sterile barrier system
	Keep away from sunlight
	Keep dry
	This way up
	Fragile, handle with care
	CE mark with notified body identification number
	Importer
	Authorised representative in Switzerland

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